0733-08
Ver. 8: 2017.Apr.10
1/3

ICSI Cumulase®

Product No.:

1612

CE

O543

Languages:

bg

cs

da

de

el

en

es

et

fr

hr

hu

is

it

kk

lt

lv

mk

nl

no

pt

ro

ru

sk

sl

sv

tr

uk

Customer Service:

E-mail: customer.service@origio.com

Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s

Knardrupvej 2, DK-2760 Måløv, Denmark

www.origio.com

Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

origio

a CooperSurgical Company

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en - english

<i>ICSI</i> Cumulase® is for the removal of the cumulus complex and corona radiata surrounding the oocyte in preparation for ICSI.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Contains Human albumin solution (HAS) Recombinant human hyaluronidase (rHuPH20) 80 U/ml

Quality control testing Sterility tested (Ph.Eur., USP) Osmolality tested (Ph.Eur., USP) pH tested (Ph.Eur., USP) Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP) HSA analysis (Ph.Eur., USP) Mouse Embryo Assay (MEA) tested
Enzyme activity
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability The products are aseptically processed and supplied sterile. Store in original container at 2-8°C, protected from light. Do not freeze. Discard excess (unused) media following warming. The product is provided in vials intended for single use. When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings Do not use the product if: 1. Product packaging appears damaged or if the seal is broken. 2. Expiry date has been exceeded.

3. The product is for single use and is not to be reused due to risk of contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Note: The product is for single use and is not to be reused due to risk of contamination.

Notes
After retrieval of the oocytes, it is recommended that the oocytes rest in equilibrated culture medium for a minimum of 2 hours before denudation.
ICSI Cumulase® is HEPES-buffered and pH-stable for use outside the incubator without prior equilibration. *ICSI* Cumulase® does not contain bicarbonate and should therefore not be placed in the CO₂ incubator without the cap on.

Instructions for use
1. Warm *ICSI* Cumulase® to 37°C for 2 hours with the cap on.
2. Fill one of the wells in a four-well dish with the contents of one vial of *ICSI* Cumulase® and the other three wells with your preferred, equilibrated culture medium.
3. Place one or several oocyte-cumulus complex(es) in the well with *ICSI* Cumulase®. After 60 seconds, gently aspirate the oocyte(s) up and down until the oocyte(s) is denuded.
4. Transfer the oocyte(s) to a well with culture medium.
5. Wash the oocyte(s) thoroughly by transferring it between several wells of culture medium.
6. The oocyte(s) is then transferred to the injection dish and placed in individual microdrops covered with equilibrated Liquid Paraffin.

It is recommended that the oocyte(s) be left in culture medium for a short period before proceeding with ICSI.

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