0734-06  
Ver.6:2014.Jun.10  
1/3

# MediCult Vitrification Cooling

Product No.:

1228

CE

O543

Languages:

bg

cs

da

de

el

et

en

es

fr

hr

hu

is

it

kk

lt

lv

mk

nl

no

pt

ro

ru

sk

sl

sv

tr

uk

Customer Service:

E-mail:  
customerservice.medicult@origio.com

Tel: +45 46 79 02 02 • Fax: +45 46 79 03 02

ORIGIO a/s

Knardrupvej 2, DK-2760 Måløv, Denmark

www.origio.com

Tel: +45 46 79 02 00 • Fax: +45 46 79 03 00

a CooperSurgical Company

**Ingredientia**  
**1222 Equilibration Medium**  
Albumini humani solutio; Aqua; CaCl<sub>2</sub>; Ethylenglycolum; Gentamicinum; Glutaminum; Glucosum; HEPES; KCl; KH<sub>2</sub>PO<sub>4</sub>; Lactas; MgSO<sub>4</sub>; NaCl; NaHCO<sub>3</sub>; NaOH; Propylenglycolum; Pyruvas;

**1223 Vitrification Medium**  
Albumini humani solutio; Aqua; CaCl<sub>2</sub>; HEPES; KCl; KH<sub>2</sub>PO<sub>4</sub>; MgSO<sub>4</sub>; NaHCO<sub>3</sub>; NaCl; NaOH; Gentamicinum; Glutaminum; Glucosum; Lactas; Pyruvas; Ethylenglycolum; Propylenglycolum; Saccharum;

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**en - english**

MediCult Vitrification Cooling is for ultra rapid cooling of human oocytes, cleavage stage embryos and blastocysts. This product is for IVF treatment of women, whether the cause of infertility is male or female. The product should only be used by professionals trained in IVF treatment.

**Products**
Vial 1: Equilibration Medium (1222)
Vial 2: Vitrification Medium (1223)

**Package**
2 x Vial 1 (1 ml); 2 x Vial 2 (1 ml)
**Contains**
Human albumin solution (HAS)
Gentamicin sulphate 10 µg/ml

**Quality control testing**
Sterility tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.5 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur, USP)
Mouse Embryo Assay (MEA) tested

**Note:** The results of each batch are stated on a Certificate of Analysis, which is available on www.origio.com.

**Storage instructions and stability**

The products are aseptically processed and supplied sterile. Store in original container at 2-8°C, protected from light. Do not freeze prior to use. Discard excess (unused) media following warming. The product is provided in vials intended for single use. When stored as directed by the manufacturer the product is stable until the expiry date shown on the vial label.

**Precautions and warnings**
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

**Caution:** The product is for single use and not to be reused due to risk of contamination.

**Caution:** All blood products should be treated as potentially infectious. Source material used to manufacture this product were tested and found non-reactive for HbsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore source material have been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

**Note:** The product contains ethylene glycol and 1,2-propanediol

**Note:** It is recommended that the blastocoele of expanded blastocysts are collapsed mechanically with micro-pipettes (PZD or ICSI) or with a laser immediately before they are incubated in equilibrium medium. The blastocoele will start re-expanding within 20-30 minutes after being collapsed. (Protocol of collapse: Vanderzwalmen et al., 2002, Hum. Reprod., 17(3), 744-751).

**Note:** Please note the need for traceability of this product. In addition national legal requirements in your country may exist in this field.

**Note:** Devices used in combination with this device should be intended for the particular purpose.

**Note:** Dispose of the device in accordance with local regulations for disposal of medical devices.

**Instructions for use**

1. Warm Equilibration Medium and Vitrification Medium to room temperature for at least 30 minutes.
2. Prepare a reservoir with enough liquid nitrogen to allow complete submersion of a goblet on a cryocane. Attach a goblet to the bottom of the cryocane and submerge in the liquid nitrogen. Place near the microscope.
3. Mix the content of the Equilibration Medium and Vitrification Medium vials by a few gentle inversions.
4. Place 1ml of Equilibration Medium and Vitrification Medium in separate wells or dishes.
5. Using a suitable pipette, transfer 2-3 oocytes or embryos/blastocysts into the Equilibration Medium. The cells initially shrink before re-expanding to their original size. Equilibration is completed once the oocytes or embryos/ blastocysts have re-expanded. The equilibration step normally takes 5 – 15 minutes.
6. Transfer the oocytes or embryos/ blastocysts in minimum volume into the Vitrification Medium (the cells shrink again). The time from transfer of the oocytes or embryos/ blastocysts into the Vitrification Medium until vitrified must not exceed 1minute
7. Quickly load the oocytes or embryos/blastocysts onto the vitrification carrier and vitrify according to Instructions for Use of the vitrification carrier.
8. Following vitrification, quickly transfer the cryocane and goblet containing the vitrification device and vitrified cells to the storage tank. Make sure that the vitrified cells are submerged under liquid

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