0739-04
Ver. 4: 2017.Apr.10
1/3

ORIGIO®
Sequential Blast™

Product No.:

8305
8306

CE

O543

Languages:

bg

cs

da

de

el

en

es

et

fr

hr

hu

is

it

kk

lt

lv

mk

nl

no

pt

ro

ru

sk

sl

sv

tr

uk

Customer Service:
E-mail: customer.service@origio.com
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s
Knardrupvej 2, DK-2760 Måløv, Denmark
www.origio.com
Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

a CooperSurgical Company

Ingredientia
8305 ORIGIO® Sequential Blast™
Acida aminica: Ala, alanylglutaminum, Arg, Asp, Asn, Cys, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val; Albumini humani solutio; Aqua; Gentamicinum; Glucosum; Hyaluronas; K₂SO₄; Lactas; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Pyruvas; SSR; Vitamini: B₁, B₂, B₃, B₅, B₇, B₉, Cholinum, Inositolum, Nicotinamidum, Pyridoxinum
8306 ORIGIO® Sequential Blast™ with phenol red
Acida aminica: Ala, alanylglutaminum, Arg, Asp, Asn, Cys, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val; Albumini humani solutio; Aqua; Gentamicinum; Glucosum; Hyaluronas; K₂SO₄; Lactas; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Phenolsulfonphthaleinum; Pyruvas; SSR; Vitamini: B₁, B₂, B₃, B₅, B₉, Cholinum, Inositolum, Nicotinamidum, Pyridoxinum

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en - english
For culture from the 4-8 cell stage through to the blastocyst stage. Can also be used for embryo transfer.
This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.
Products 8305 ORIGIO® Sequential Blast™ 8306 ORIGIO® Sequential Blast™ with phenol red
Contains Human albumin solution (HAS) Recombinant human insulin Gentamicin sulphate 10 µg/ml
Quality control testing Sterility tested (Ph.Eur., USP) Osmolality tested (Ph.Eur., USP) pH tested (Ph.Eur., USP) Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP) Mouse Embryo Assay (MEA) tested HSA analysis (Ph.Eur., USP) Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.
Storage instructions and stability The products are aseptically processed and supplied sterile. Store in original container at 2-8 °C, protected from light. Do not freeze. Discard excess (unused) media following warming. The product is to be used within 7 days after opening. When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.
Precautions and warnings Do not use the product if: - Product packaging appears damaged or if the seal is broken. - Expiry date has been exceeded. - The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.
Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.
Caution: This product contains gentamicin and should not be used on patients that have a known allergy to gentamicin or similar antibiotics.
Note: Please note the need for traceability of this product. In addition, national legal

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requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instrucciones de uso
- Equilibrate for a minimum of 2 hours in 5-6% CO₂ at 37°C prior to use. The time required to achieve full equilibration depends on the volume of medium and oil overlay. - At the 4-8 cell stage the embryos are carefully washed in pre-equilibrated ORIGIO® Sequential Blast™ and transferred to fresh microdrops or wells of ORIGIO® Sequential Blast™. - The embryos should be moved to fresh drops/wells of ORIGIO® Sequential Blast™ every other day until blastocyst formation at Day 5/Day 6.

Embryo transfer
- The embryos are prepared and transferred to the uterus in pre-equilibrated fresh ORIGIO® Sequential Blast™ or a preferred transfer medium. - Flush the transfer catheter with the chosen transfer medium prior to use.

Each laboratory should make its own determination of which medium to use for each particular procedure.

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