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en - english

For isolation of viable spermatozoa by the density gradient method.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.8 EU/ml (Ph.Eur., USP)
Sperm survival tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warming.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:

- Product packaging appears damaged or if the seal is broken.
- Expiry date has been exceeded.
- The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use
1. For custom gradient preparation, use the formula below to prepare a chosen gradient, using SupraSperm® 100 and Sperm Preparation Medium.

X ml SupraSperm® 100
Y ml Sperm Preparation Medium

Gradient % =

X
X
+
Y

 x 1.0 x 100 %

- For each semen sample, prepare a separate gradient for each 1 ml volume. Prepare gradients by using 1-2 ml of each custom-made SupraSperm® solutions and pre-equilibrate in a CO₂ environment at 37°C. *Gradients should be prepared immediately before use for optimal results.*
 - The semen sample is thoroughly mixed (i.e. repeated tilting for 20 minutes at room temperature).
 - After the mixing process is complete, sperm concentration and motility should be assessed.
 - Carefully dispense 1 ml of liquefied semen sample on top of the prepared gradient.
- Adding too much sperm will result in overloading and poor separation.*
- The gradient is centrifuged at 300 g for 20 minutes.
 - Remove the supernatant from the pellet and place the pellet in a clean centrifuge tube.
 - Re-suspend the pellet in 5 ml of pre-equilibrated Sperm Preparation Medium and centrifuge again at 300 g for 10 minutes. Aspirate the supernatant.
- Repeat this washing procedure.*
- Add a small amount of pre-equilibrated Sperm Preparation Medium and determine motility and concentration of spermatozoa in the washed sample.
 - Finally, re-suspend the washed sperm in a suitable volume of Sperm Preparation Medium.
- When the caps of the tubes are tightened, the prepared semen can be kept at room temperature (20-25°С) for up to 1 hour prior to insemination. It is recommended that the sperm sample be wrapped in aluminium foil. Alternatively, the unwrapped sperm sample can be stored in a CO₂ environment at 37°С.*

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