Embryo Thawing Pack

Product No.:

1098

0543

Languages:

bg	cs	da	de	el
en	es	et	fr	hr
hu	is	it	kk	lt
lv	mk	nl	no	pt
ro	ru	sk	sl	sv
tr	uk			

Customer Service:

E-mail: customer.service@origio.com
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s
Knardrupvej 2, DK-2760 Måløv, Denmark
www.origio.com
Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

a CooperSurgical Company

Ingredientia

1041 Embryo Thawing Pack – Vial 1
Albumini humani solutio; Aqua; CaCl₂; KCl; KH₂PO₄; MgCl₂; NaCl; Na₂HPO₄; Propylenglycolum; Saccharum; SSR

1042 Embryo Thawing Pack – Vial 2
Albumini humani solutio; Aqua; CaCl₂; KCl; KH₂PO₄; MgCl₂; NaCl; Na₂HPO₄; Propylenglycolum; Saccharum; SSR

1043 Embryo Thawing Pack – Vial 3
Albumini humani solutio; Aqua; CaCl₂; KCl; KH₂PO₄; MgCl₂; NaCl; Na₂HPO₄; Saccharum; SSR

1179 Embryo Thawing Pack – Vial 4
Albumini humani solutio; Aqua; CaCl₂; KCl; KH₂PO₄; MgCl₂; NaCl; Na₂HPO₄; SSR

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en - english

For thawing of human zygotes and cleavage stage embryos frozen using the Embryo Freezing Pack.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Package
1041 Embryo Thawing Pack (Vial 1)
1042 Embryo Thawing Pack (Vial 2)
1043 Embryo Thawing Pack (Vial 3)
1179 Embryo Thawing Pack (Vial 4)

Pack size
10984002: 1 x Vial 1 (2 ml); 1 x Vial 2 (2 ml); 1 x Vial 3 (2 ml); 1 x Vial 4 (2 ml)
10984010: 1 x Vial 1 (10 ml); 1 x Vial 2 (10 ml); 1 x Vial 3 (10 ml); 1 x Vial 4 (10 ml)

Contains
Human albumin solution (HAS)
Recombinant human insulin

Quality control testing
Sterility tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0,1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warming.
The product is provided in vials intended for single use.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: The product is for single use and is not to be reused due to risk of contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Note: Do not place in the CO₂ incubator, as the buffer capacity of the phosphate buffered Embryo Thawing Pack is insufficient for the CO₂ atmosphere.
The cryo-preserved zygotes or embryos are osmotically stressed and should be handled very carefully.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- Pre-warm the Embryo Thawing Pack for a minimum of 2 hours at room temperature.

- Remove the straws from the liquid nitrogen (LN₂) and keep at room temperature for 30 seconds. Then place straws in water at 30°C for 1 minute.
- Open the straws or cryo-tubes according to the manufacturer's instructions and expel the contents into a Petri dish.
- Place zygotes or embryos in the medium from Vial 1 at room temperature for 5 minutes.
- Transfer zygotes or embryos into the medium from Vial 2 at room temperature for 5 minutes and then into the medium from Vial 3 at room temperature for 10 minutes.
- Transfer zygotes or embryos into the medium from Vial 4 and wash in an appropriate culture medium, depending on the stage of development at freezing and the day of transfer. Place the zygotes or embryos in a culture using the same medium.

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